

LG 1-2-3 PRO-DORSAL ORTHOSIS INSTALLATION INSTRUCTIONS



Give More Care for LESS COST ... The LEEDer way

Features Flex-E-Core Technology for EASY FITTING. Bend it to fit!



1. Open all Velcro Closures and place resident's hand (fingers and thumb) on palmar aspect of splint.
2. Check position of hand and thumb; adjust as necessary to fit resident by bending the palmar and /or thumb aspects AS NECESSARY to comply with the patient's condition. Use a firm surface to adjust the palmar and thumb aspect of the splint.
3. Check the wrist to ensure the necessary degree of extension. Adjust as necessary.
4. Fasten the Velcro closures over the wrist (Apply Strap # 1). Check that the strap is secure and not over tight.
5. Fasten the Velcro closures over the forearm. (Strap Marked # 2) Check that the strap is secure and not over tight.
6. Bring the hand strap UNDER the palmar portion of the splint and wrap up and around to the dorsal aspect of the splint (Strap Marked #3). Fasten Velcro closure on top.
7. Wrap the thumb strap comfortably around the thumb to secure in place (Strap Marked # 4).
8. Apply Finger Separator as desired.
9. Recheck the Hand, Fingers and Thumb for correct positioning.