



## Sizing Chart MULTI USE BOOT

### LMUA: Multi Use Boot: Adult, Pediatric and Firm

Size	Measurement	Order Code
ADULT	ADULT SHOE size 5 or greater	<b>LMUA</b>
Pediatric	ADULT SHOE size under 5	<b>LMUP</b>
FIRM	ADULT OVER 225 LBS.	<b>LMUF</b>

### SIZING PROTOCOL FOR LSO

### LUMBO-SACRAL ORTHOSIS: LSO –VCO LG-LSO-CF

ALL UNITS FIT UP TO 50 INCH GIRTH. FOR LARGER GIRTHS USE THE TWO EXTRA STRAPS INCLUDED.

1. Sizing choice dependent upon body size and gender. All sizes fit up to a 50-inch abdomen girth.
2. Choose size based upon desired objective: the larger units cover and wrap like a body jacket; the smaller units provide more localized support.
3. Women tend toward small units with small front panels to fit below bosom.

Select the **Anterior (FRONT)** and **Posterior (REAR)** panel size that is appropriate. The panel dimensions given below are the vertical panel height.

Size	Posterior Anterior Panel	Order Code
Small	11 inch – 6 inch HEIGHTS	<b>11-6 P or T</b>
Medium	11 inch – 9 inch HEIGHTS	<b>11-9 P or T</b>
Large	16 inch – 9 inch HEIGHTS	<b>16-9 P or T</b>

**P= Padded and T= Thin panel is about ¼ inch thinner with rip-stop nylon durable exteriors.**

The Flex-E-Core panels are MOLDABLE without tools or heat. They are comfortable for better patient compliance making your work successful. Custom fit each panel to accommodate your patient.