

HKO Air Abductor Instructions

Order Code = **LHKO AIR**

- **Open cuffs; wrap thighs and fasten Velcro**
- **Use Hand Pump to inflate-deflate AIR-bladder**
- **Extra strap length may be cut off if not necessary**
-



Perfect for Knee Scissoring patients. Now with longer straps to allow single or bi-lateral HKO / Knee splinting combination.

- Easy to adjust Flex-E-Core in 2 sizes: 28 inch and 32 inch.
- Large Air Bladder and Easy Hand Pump
- Easy don / doff
- Mechanical Spreader Medicare code requirement: open Velcro closures and remove Air-Bladder Flex-E-Core inserts and replace with white, KYDEX spreader inserts.
- Adjust Range of Spreader before inserting: Rotate one KYDEX spreader insert counterclockwise to open and clockwise to close. Install inserts and close Velcro.
- Regularly Check for patient comfort and contra-indications.

ORDER CODE = LHKO AIR

Toll Free Customer Service 866-814-0192

www.LEEDerGroup.com

Toll Free FAX 866-818-0373

LEEDerGroup@LEEDerGroup.com

8508 NW 66th St Miami FL 33166