

# LEEDerGroup.com

## 1-2-3 PRO GRIP

8508 NW 66th Street  
Miami, FL 33166  
T: 305.436.5030

Single Patient Use Only

RX Only—Custom Fit

Before using this device, **PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY.** Correct application is vital to the proper functioning of the de-

### INTENDED USE:

Joint Contracture Wrist, Hand and Fingers  
Wrist Drop

### CONTRAINDICATIONS:




Swelling and / or Redness

### WARNINGS & PRECAUTIONS:


Application by trained medical personnel only.  
Should be removed **AT LEAST** once every shift for assessment.  
Should be **REMOVED** and the physician notified if:  
A. persistent redness and irritation of skin occur  
B. excessive pain and / or swelling occur.

### FITTING INSTRUCTIONS:


1. Remove the Orthosis from the package and determine which of the hand rolls will be used (small, medium or large). Keep Extra Parts in the mesh laundry bag for future use.
2. Unfasten all Hook and Loop closures.
3. (Picture 2) For extreme contracture initially use the Palmar T-Bar without Hand rolls. As the contracture weakens, add Hand Roll to increase the Palmar opening. To do so, slide Hand roll over T-Bar. You may combine Hand rolls to vary the size.
4. Assess the degree of wrist contracture. Bend the Splint to the appropriate position to fit the degree of wrist contracture and/or deviation. Flex-E-Core is stiff to resist tone; bend as necessary.
5. Place the hand on the splint. (Picture 3) Check the wrist to assure a proper fit. Re-adjust as necessary. Bend to conform to the forearm.

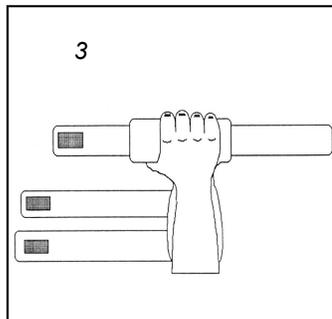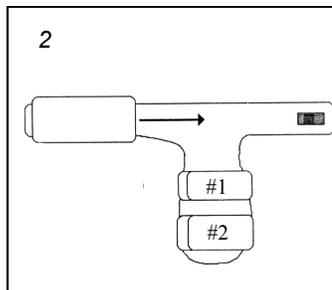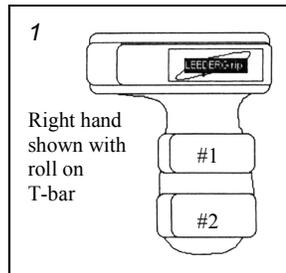


7. STRAP: Apply Strap #1 (Picture 4). Fasten the hook and loop closure over the wrist. Check that the strap is secure but not too tight by placing one finger between the arm and the strap.

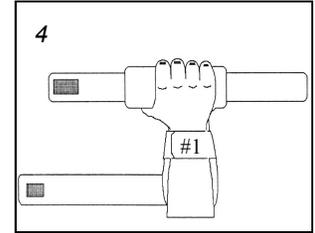

8. STRAP: Apply Strap #2 (Picture 5) Fasten the hook and loop closure over the forearm. Wrap the strap around and fasten. Check that the strap is secure but not too tight by placing one finger between the hand and the strap.

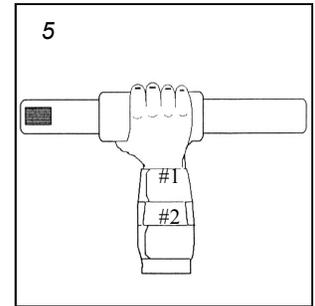

9. STRAP: Apply Strap #3 (Picture 6) Fasten the hook and loop closure over the fingers. Wrap the straps together. Check that the strap is secure but not too tight.

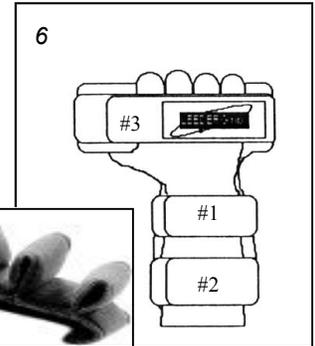

10. Assess the hand for adverse reactions frequently (at least every hour initially). Thereafter, remove the splint and assess the hand at least once every shift.

**\*Finger Separator (included)** may be applied if required. Fasten to the Grip Roll with Velcro. **Fingers go between panels; DO NOT put fingers inside the loops.**

### CLEANING INSTRUCTIONS:

**Note: refer to facility infection control regulations**  
Remove cover from splint. Fasten all Hook and Loop closures. Place in mesh laundry bag provided. Machine wash in warm water and rinse thoroughly. Tumble dry **NO HEAT**.

### Flex-E-Core Technology: Adjust it anyway you like!

Clean with mild detergent and rise thoroughly. Completely dry with a soft cloth

### WARRANTY:

Product warranty ninety (90) days from date of shipment as to product quality and workmanship. Manufacturer's written warranty is given in lieu of any implied warranty, including warranty of merchantability or fitness for a particular purpose.

### CAUTION:

This Orthosis device provided by or on the order of a licensed Health Care Professional.

LATEX FREE



### Part No.

LGSL PRO (small left), LGSR PRO (small right)  
LGML PRO (medium left), LGMR PRO (medium right)  
LGLL PRO (large left), LGLR PRO (large right)  
LGPU PRO Pediatric universal

V.1.16GS

Email: [info@LEEDerGroup.com](mailto:info@LEEDerGroup.com)