

Give More Care for LESS COST ... The LEEDer way

Instructions: AmBoot Fast Lock Donuts



"Fast Lock" rubber "Donuts" pop out easily with pen or pencil



- Use a pencil or pen to pop the rubber donuts off from around the retaining nuts on the bottom of the boots; small cut-outs are provided.
- Unscrew the retaining nuts to remove the amb-pad
- Simply reverse this procedure to re-attach the amb-Pad
- Press the donuts back around the nuts after tightening; keep the cut-outs to the back

Double Positive Connection Points. No tools Required.